

ProPACT for the Steris System 1™ ANSWERS
Provided by Raven Biological Laboratories



Answers:

True/False

- 1 True DO NOT refrigerate ProPACT.
- 2 True ProPACT should be stored in a cool, dry place (NOT refrigerated) and away from sterilants and out of direct sunlight or UV rays.
- 3 True The test is Negative only if the control has growth and the test does not.

Multiple Choice:

- 1 E Incubation is 24 hours. However, results will be valid and identical beyond 24 hours if for some reason (weekend, holiday, illness) results cannot be read.
- 2 F Any color change in the culture media to or toward yellow and/or any turbidity indicates a positive result.
- 3 E Record Results according to your protocol if test is negative.
- 4 F If test culture is positive for growth DO NOT assume human error. If repeat test culture is Positive, contact sterilizer manufacturer.