



## Raven Labs

A division of Mesa Laboratories, Inc.

8607 Park Drive

Omaha, NE 68127 USA

PH: 1-800-728-5702 or 1-402-593-0781

Email: [info@ravenlabs.com](mailto:info@ravenlabs.com)

[www.ravenlabs.com](http://www.ravenlabs.com)

### Raven ProSpore Ampoule Self-contained Biological Indicator

For 121°C steam sterilization cycles only. Store in a refrigerator at 2-8°C. **DO NOT FREEZE.**

**Description:** Raven ProSpore Ampoule is intended for use in the monitoring of saturated steam sterilization cycles at 121°C. Each ProSpore Ampoule contains a spore suspension of *Geobacillus stearothermophilus* within a growth medium also containing Bromocresol Purple to function as a pH indicator. The acid production associated with growth causes a change in color from purple to or toward yellow.

For greatest control of sterilized goods we recommend that Raven ProSpore Ampoule Biological Indicators be included with every load. Do not use damaged ampoules. Do not use after expiration date. Since ProSpore contains live cultures, ampoules should be handled with care. ProSpore is not intended for flash sterilization processes.

**Disposal:** Sterilize all positive and expired units prior to disposal.

### Instructions for Use

**Exposure:** Place one or more Raven ProSpore Biological Indicators in the most difficult location to sterilize, usually near the drain or suspended in a volume of liquid. Run cycle.

**Caution:** After sterilization, handle ampoules with care. Contents of the ampoule are hot and under pressure. Failure to allow sufficient cooling time (10-15 minutes) may result in bursting of the ampoule.

**Incubation:** Place the processed ampoule in a vertical position in an incubator at 55-60°C. Mark a control ampoule as such and incubate along with processed ampoules to ensure spore viability. Raven recommends an incubation time of 48 hours.

**Monitoring:** Examine the ProSpore ampoules daily during incubation. Record observations. All positive ampoules should be recorded and then disposed of immediately.

### Interpretation:

**Control:** The control ampoule should exhibit a color change to or toward yellow and/or turbidity. If the control ampoule does not show signs of growth, consider the test invalid.

**Test:** A failed sterilization cycle is indicated by turbidity and/or a change in color to or toward yellow. A test ampoule that retains its purple color indicates an adequate sterilization cycle.