



Raven Labs

A division of Mesa Laboratories, Inc.

8607 Park Drive

Omaha, NE 68127 USA

PH: 1-800-728-5702 or 1-402-593-0781

Email: info@ravenlabs.com

www.ravenlabs.com

Raven Spore Wires/Threads

Instructions for Use

1. Aseptically place the wires/threads into the sterilization load. Spore wires/threads are designed to be used in areas with small lumen or areas where a spore strip will not fit properly.
2. Run the sterilization cycle as normal.
3. Aseptically remove the wires/threads from the sterilization load and aseptically (use of laminar flow hood or similar equipment) transfer the spore wire/thread to a sterile tube of growth media.
4. Incubate for 7 days at 30-35°C for EtO/Dry Heat or at 55-60°C for Steam/H₂O₂. Daily checks should be made for Growth/No Growth and results documented.